


Manage translations for subject initiated events

1 Overview

- It is possible to translate the forms in the subject initiated events, as well as the respective reminders, by following the steps below:
 - Choose the language(s) the form(s) will be translated to.
 - Export the excel file containing the list of all items in the form to be translated.
 - Perform the translations.
 - Import the file containing the translation.
- Please note that Viedoc provides only support for exporting the text for translation and importing the translated file(s). The translation itself is performed outside Viedoc.


2 Set the language(s) for translation

- It is possible to define the default language that was used when setting up the study design, as well as the languages the forms will be translated to.
 - In Viedoc Designer, go to the **Design Settings** section, under the **Details** tab.
 - In the **Languages** section, set the following:
 - Default** - choose the default language used in your design (the language used when designing all the events, activities, forms and items). This is the language to translate from.
 - Additional** - click and select the languages to translate to.
 - Click **Save changes**.

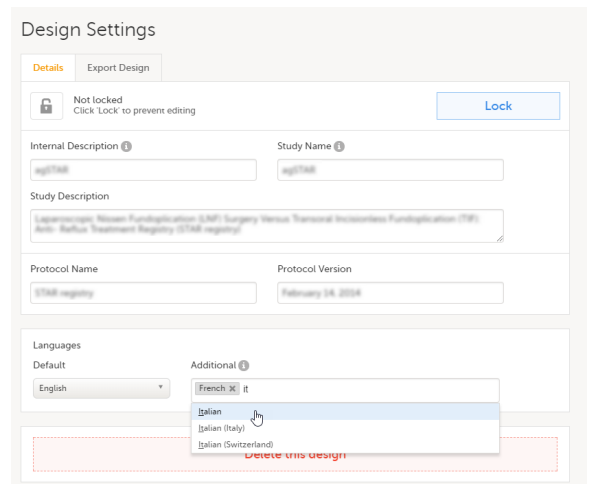

3 Export the form(s) text for translation

- In Viedoc Designer, in the **Design Settings** section, under the **Language Import/Export** tab, in the **Export language texts for translation** pane:
 - Select the language to translate to from the drop-down list. The available languages in the list are the ones selected as **Additional** at the previous step.
 - Click and select the forms to be translated, one by one. Make sure that you select the forms included in subject initiated events. The form named **Workflow** contains all the reminders set up in the study workflow.
 - Click **Export**. You will get an excel file with one separate sheet for each selected form, containing all the texts for the respective form.

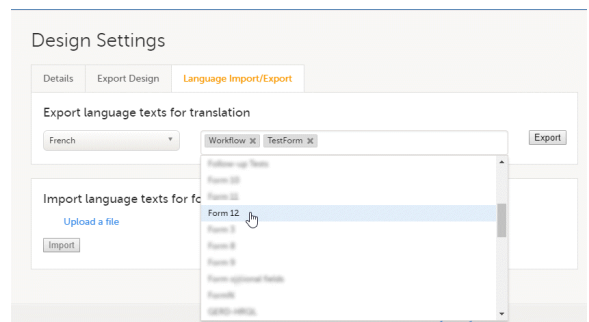

4 Perform the translation

- In the excel file obtained at the previous step, add the translated text in the column to the right. Make sure that only the text is translated and the tags are kept exactly as in the **Default text** column.

	A	B	C	D
1	ID	Category	Default Text	fr
2	MU	Form Name	Medication use	Utilisation de Médicaments
3	MU	Form Description		
4	id_10216_2	Group Label	Concomitant Medications / Therapies	Médicament concomitants
5	id_10216	Form Label	With other Medications / Therapies?	en-Therapie?
6	id_10216	Form Choice 1	Yes	Oui
7	id_10216	Form Choice 2	No	Non
8	id_10216	Form Label	When other Medications / Therapies used outside of the Presc and Dispensing?	When other Medications / Therapies used outside of the Presc and Dispensing?
9	id_10216	Form Label	Medications tag: Open	Medicament-Tag: Open
10	id_10216	Form Label	Medications tag: Close	Medicament-Tag: Close
11	id_10216	Form Label	Has the subject used any Medications / Therapies in the last 24 hours?	Has the subject used any Medications / Therapies in the last 24 hours?
12	id_10216	Form Label	Has the subject used any Medications / Therapies in the last 7 days?	Has the subject used any Medications / Therapies in the last 7 days?

5 Import the translated file

- When the translation is performed and you have the excel file, you have to import it by following the steps below:
 1. Go to the **Design Settings** section, under the **Language Import/Export** tab.
 2. Under **Import language texts for form translation**, click **Upload a file**.
 3. Browse and select the excel file that contains the translation.
 4. Click **Import**.

6 Important

- If you have performed the translation while working on the training server and intend to import the design to the production server afterwards, in order to have the translations available after the import you have to repeat the following operations on the production server as well:
 - add the additional language(s), as described in section *Set the language(s) for translation* above.
 - import the translation file(s) again, as described in section *Import the translated file* above.
- If you revise your design (e.g. by changing texts that were translated or item IDs) after having performed the translation, make sure to update the translations accordingly and re-import the translated files.